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#### **EN - Safety instructions for disposable respirators**

The respirator is designed to protect the respiratory system from bacteria, aerosols, dust, smog particles, and pollen. It is not intended for protection against chemicals, toxic gases, or other specific hazards unless specified in the manufacturer's instructions. Use the respirator only in environments where respiratory protection from these particles is required. Use the respirator only for the purposes specified in the instructions and ensure that it is suitable for protection against specific types of

hazardous particles. Before use, check that the respirator is not damaged. Ensure that the respirator fits properly on your face to prevent air leaks around the edges. This respirator is for single-use only. After use, dispose of it immediately. Do not reuse. If the respirator becomes contaminated, damaged, or wet during use, replace it with a new one immediately. If you experience any discomfort, such as shortness of breath, headache, dizziness, or skin irritation while using the respirator, stop using it immediately and seek medical attention. If you have any health issues, such as asthma or other respiratory conditions, consult a doctor before using the respirator. Store the respirator in a dry, cool place, away from direct sunlight, to ensure its integrity before use. The respirator is not intended for children unless explicitly stated in the manufacturer's instructions. After use or expiration of the shelf life, dispose of the respirator in accordance with local waste disposal regulations. If you have any doubts about the proper use, safety, or suitability of the respirator for specific conditions, always follow the manufacturer's instructions or consult an expert.

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